


medipal

Fearless innovation against infection

WE ARE MEDIPAL, PART OF PAL INTERNATIONAL.

We make work places and care environments more hygienic and safer places to be.

Our fearless innovation is leading the fight against germs and infection.

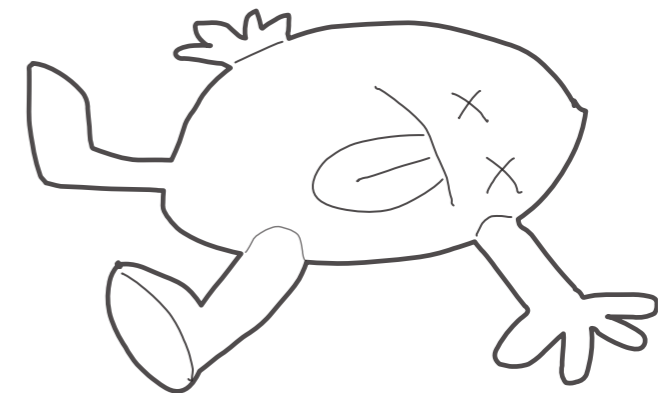

WHAT DOES THAT MEAN?

It means we make the best possible products.


It means we invest £millions in regulatory compliance, quality systems and testing so you can trust our products.

It means we understand the real-life challenges on a shift.

It means we support nurses all around the world with free training.



**Health Care
Associated Infections
are the biggest threat
to successful patient
outcomes today**

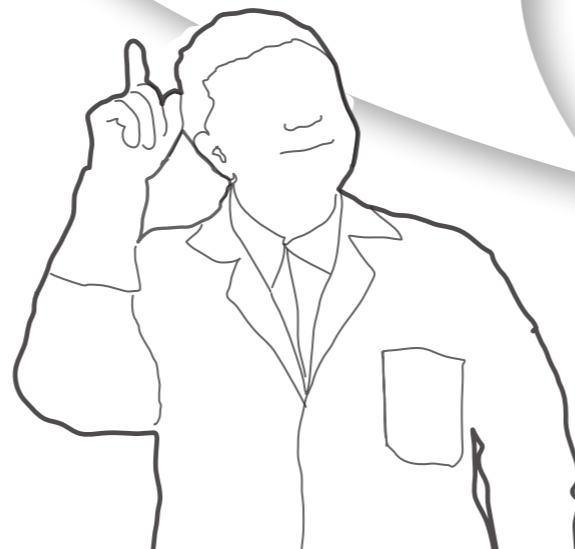

Dr. Neil Wigglesworth,
Director Infection Prevention and Control,
Guys & St Thomas' Foundation Trust.
Immediate Past President of the Infection
Prevention Society.

HEALTH CARE ASSOCIATED INFECTIONS: THE FACTS






HCAIs cost the NHS and HSE over £1 billion per year

8 in 100 patients develop a HCAI




7000 deaths every year caused by HCAIs

Patients stay in hospital longer, suffering pain
or in some cases, long term disability

Everyone loses confidence in the NHS and HSE

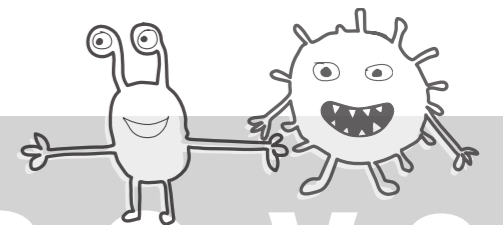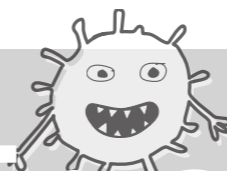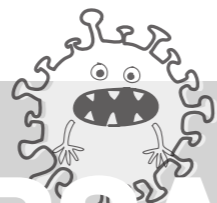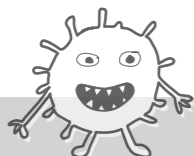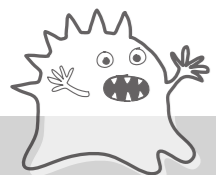

C-difficile/MRSA/Norovirus/CPE/SARS-CoV-2

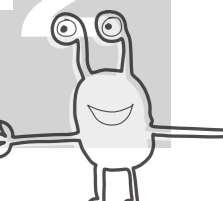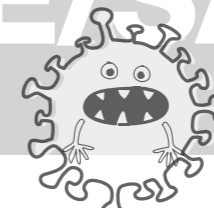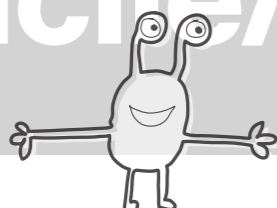


REGULATORY COMPLIANCE, REAL WORLD EFFICACY

EN16615 is the European Standard for chemical disinfectants.

It's also known as the 4 field test.

Our products all exceed the standards required, and with short contact times practical in the real world.

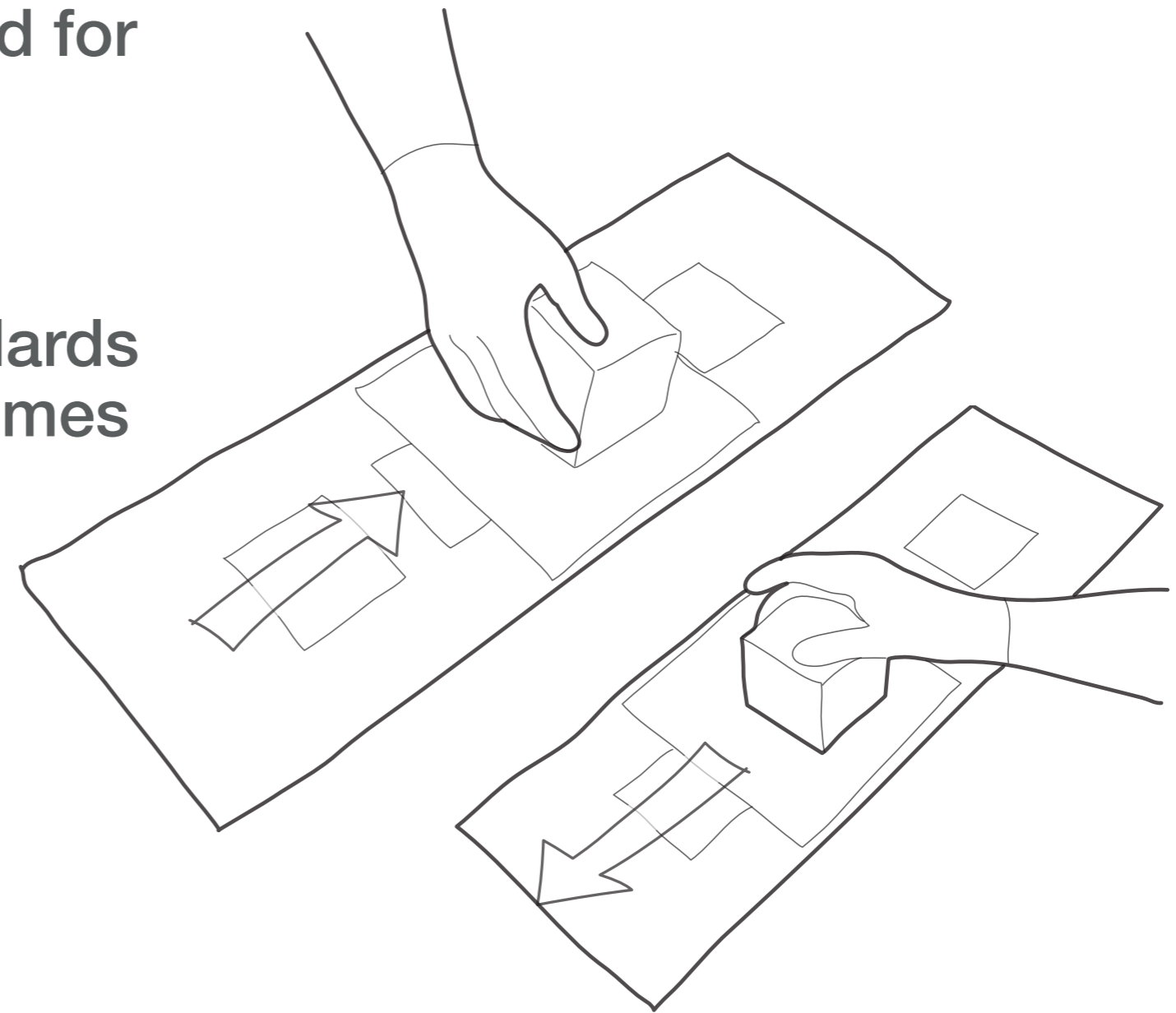

MEDIPAL - THE RANGE



A complete range for all requirements


MEDIPAL DISINFECTANT WIPES

- ✓ **Clean**
- ✓ **Disinfect**

- ✓ **Alcohol free**
- ✓ **PHMB free**
- ✓ **Ready to use**
- ✓ **Ultra low linting Tritex® material**



EFFECTIVE IN 1 MINUTE

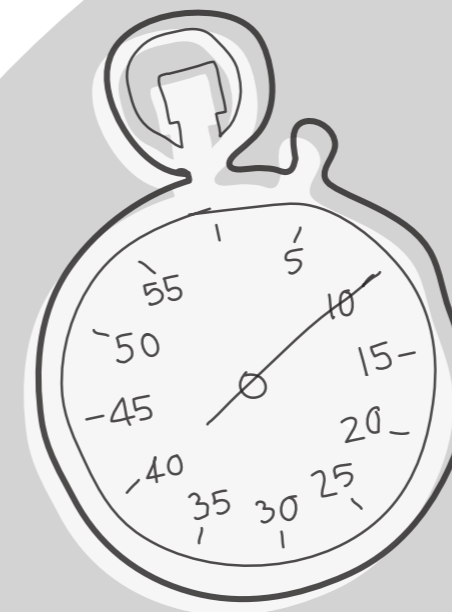

- ✓ **C-difficile**
- ✓ **Norovirus**
- ✓ **MRSA**
- ✓ **CPE**
- ✓ **Coronavirus**

MEDIPAL DISINFECTANT WIPES

Formats for every environment



MEDIPAL DISINFECTANT “COMBINATION CHEMISTRY”

Powerful double action, effective against viruses and bacteria.

- ✓ Triamine kills micro-organisms by breaking down and altering molecules
- ✓ Quats dissolve the cell wall, causing cells to split open and die

Working together, pathogens are torn open, poisoned and destroyed, as well as removed by the ‘S’ shaped wiping action. All at once. Gone in 1 minute.



MEDIPAL DETERGENT WIPES

- ✓ Multi-surface cleaning
- ✓ Great cleaning power
- ✓ Low smear
- ✓ Alcohol free
- ✓ Neutral pH

Detergent wipes perfect for multi-surface cleaning and damp dusting prior to disinfection.

- ✓ Unique Tritex[®] material clings to dust for maximum efficiency



MEDIPAL DETERGENT WIPES

Formats for every environment




MEDIPAL ALCOHOL DISINFECTANT WIPES

- ✓ **Disinfects**
- ✓ **Kills 99.999% of bacteria**
- ✓ **Optimal IPA level (70%)**
- ✓ **Smear free**
- ✓ **Ready to use**
- ✓ **Ultra low linting Tritex® material**

**Effective
against
the causes
of TB**

EFFECTIVE IN 1 MINUTE

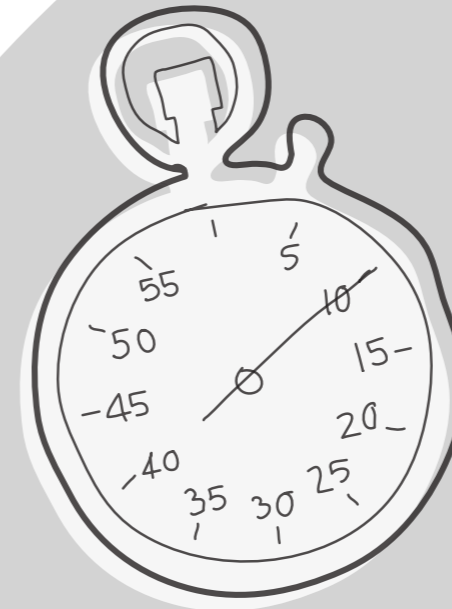

- ✓ **Bacteria, Mycobacteria and Yeasts**
- ✓ **Kills a range of micro-organisms including ESBL, CPE & MRSA**

MEDIPAL ALCOHOL DISINFECTANT WIPES

Flexible formats




MEDIPAL CHLORINE DISINFECTANT WIPES

- ✓ **High Level Disinfection**
- ✓ **Stable formula**
- ✓ **Ready to use: no mixing or activation needed**

**1 minute contact
time against SARS-
CoV-2 the virus that
causes COVID-19**

**EFFECTIVE IN
1-2 MINUTES**

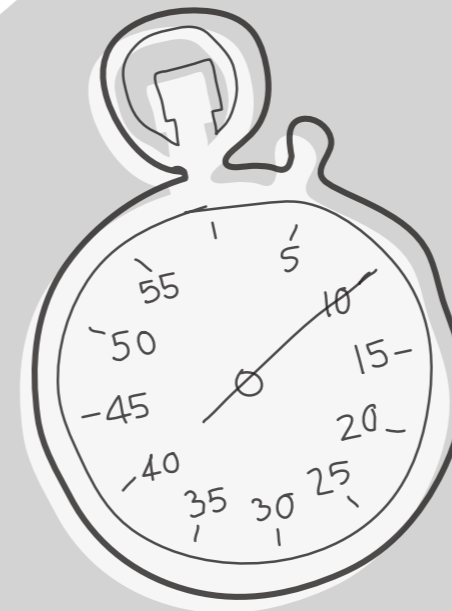

- ✓ **Sporicidal**
- ✓ **Bactericidal**
- ✓ **Fungicidal**
- ✓ **Virucidal**
- ✓ **Mycobactericidal**

MEDIPAL CHLORINE DISINFECTANT WIPES

- ✓ Stable packaging format: soft pack of 50 wipes
- ✓ No need to dilute or activate – ready to use.
- ✓ No goggles or respiratory PPE required



MEDIPAL SINGLE SACHET WIPES

Alcohol Surface Wipes with 2% Chlorhexidine

Effective against 99.99%
bacteria and yeasts on
non-invasive medical devices

Box of 200 individual wipes

2% Chlorhexidine in 70% Alcohol Skin Wipes

Ideal for plaster residue
removal

Box of 200 individual wipes

Alcohol Disinfectant Wipes

Gamma irradiated after
manufacture to ensure
extra bio-security

Box of 100 individual wipes



MEDIPAL SKIN CLEANSING WIPES



- ✓ Dermatologically tested – safe on skin
- ✓ Cleanse, hydrate and moisturise
- ✓ Skin friendly pH
- ✓ Use hot or cold


- ✓ Refreshing
- ✓ De-odourising
- ✓ Rinse free
- ✓ Waterless bathing
- ✓ Microwave and warmer friendly
- ✓ Alcohol, lanolin and Paraben free

MEDIPAL SKIN CLEANSING WIPES

✓ 8 pack – ideal for full body coverage



8 STEP BED BATHING TECHNIQUE

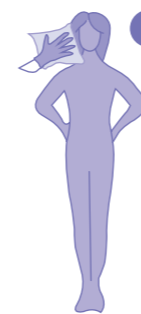

1 Head & Neck
Use the wipe on face, paying particular attention to nose, ears and neck but avoid eyes.



2 1st Arm & Armpit
Starting from upper shoulder, clean down first arm and hand.
Pay particular attention to areas between the fingers, elbow joint, and any skin folds.
Clean under armpit.



3 2nd Arm & Armpit
Starting from upper shoulder, clean down second arm and hand.
Pay particular attention to areas between the fingers, elbow joint, and any skin folds.
Clean under armpit.



4 1st Leg & Foot
Starting from hip joint, clean down first leg to the foot and sole.
Pay particular attention to areas between the toes, knee joint and any skin folds.



5 2nd Leg & Foot
Starting from hip joint, clean down second leg to the foot and sole.
Pay particular attention to areas between the toes, knee joint and any skin folds.



6 Chest & Abdomen
Clean the chest being careful with the sensitive areas.



7 Back & Lower Back
Clean the back and lower back.



8 Groin & Pubic Area
Wash the groin and perineal area following your standard nursing practice.


MEDIPAL WIPES

Made with Tritex[®] for the very best performance


- ✓ 3 non-woven layers make a 3D web of high performing coated fibres
- ✓ Higher strength, even when wet.
- ✓ Even moisture distribution. No liquid pooled in the bottom of the pack
- ✓ Perfect moisture transfer – no soapy residue left behind
- ✓ Ultra low linting to avoid microscopic fragments creating a contamination hazard

IT'S ALL IN THE ACTION.

- ✓ Follow your PPE policy
- ✓ Use just one wipe at a time
- ✓ Don't scrunch – use flat
- ✓ S shape action, clean to dirty
- ✓ Fully wet the surface and allow to dry naturally.

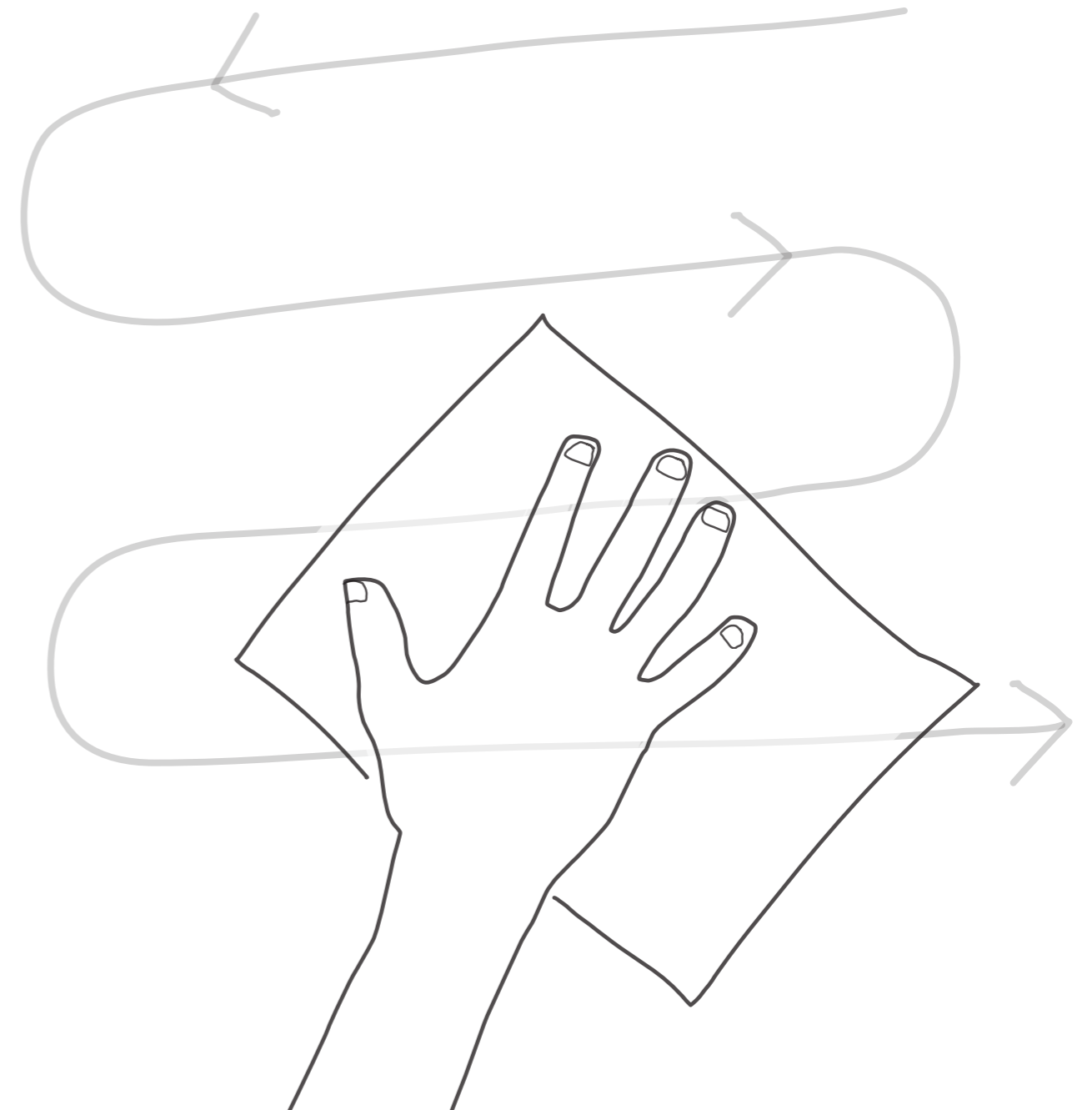


Tritex[®] Wipe

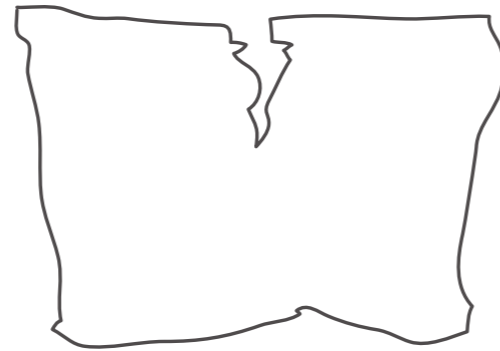

- ✓ Hard to tear
- ✓ Ultra low linting
- ✓ Super absorbent, no pooling
- ✓ Stays strong when wet
- ✓ Even spread of moisture = consistent results
- ✓ Strong in all directions

Non Tritex[®] Wipe

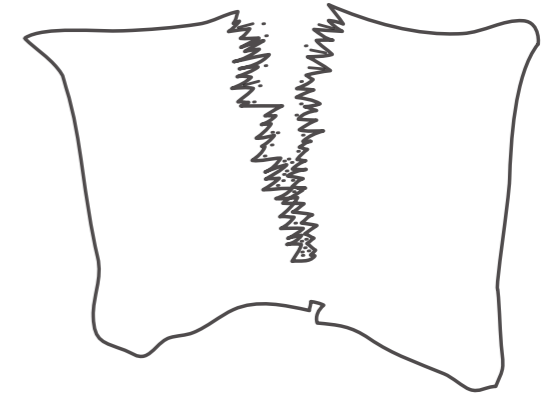


- ✓ Easy to tear
- ✓ High linting
- ✓ Less absorbent, liquid pools in pack
- ✓ Weaker when wet
- ✓ Wetter areas of wipe = inconsistent results
- ✓ Weaker, stretchy especially in one direction

INDICATOR PRODUCTS

Medipal indicator products help communication, avoiding mistakes or duplication.

- ✓ **Clean and Dirty** Indicator Notes available in pads of 250 notes, 4 pads in a box.
- ✓ **I am clean** indicator tape also available on a roll.

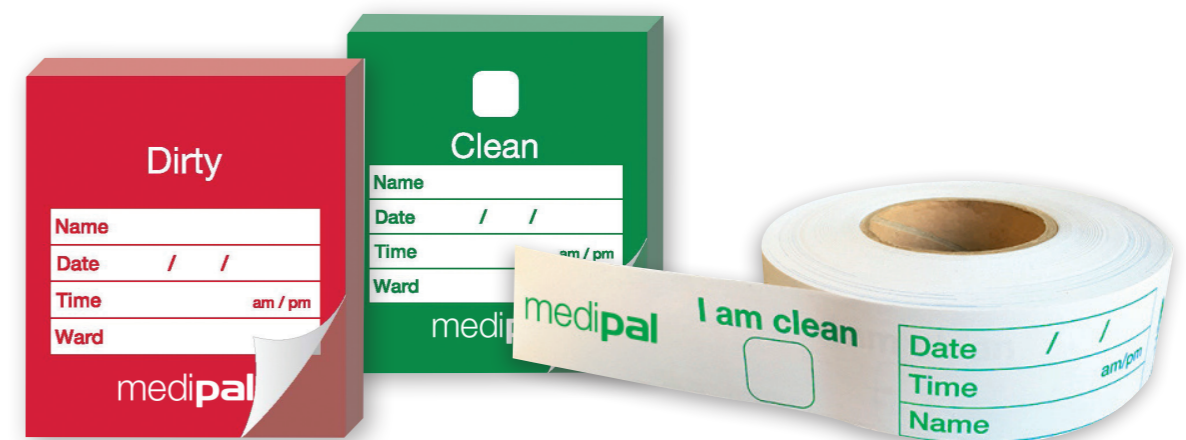


MEDIPAL CLIP ON DISPENSER

- ✓ Holder for Medipal soft packs
- ✓ Suitable for 50 and 100 wipe packs
- ✓ Greater convenience, less waste
- ✓ Clips straight on to beds and trolleys.



EASY FIT

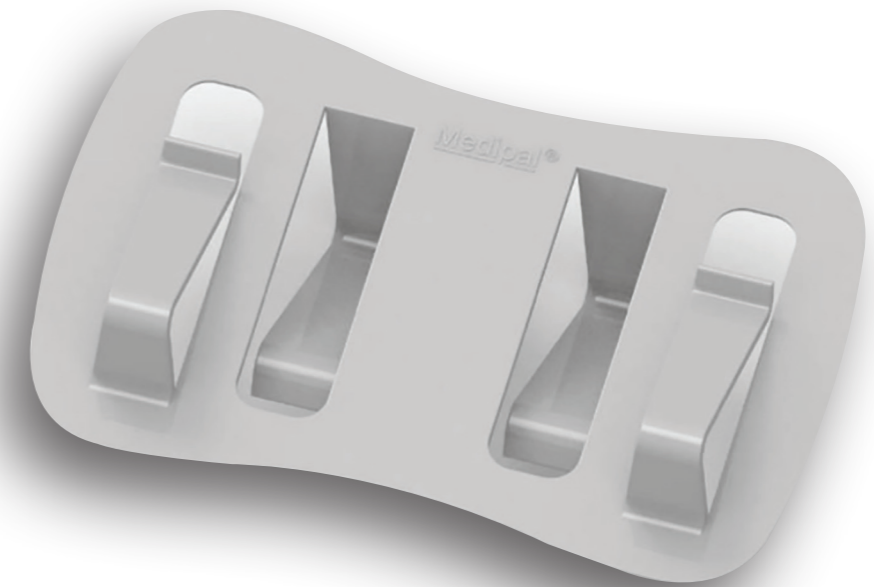


SUPPORTING NURSES WITH FREE TRAINING

- ✓ Free CPD accredited Learning Unit
- ✓ Created for nurses by Medipal and Nursing Times
- ✓ Understand risks, develop practical strategies to prevent and control infection

Get started at medipal.co.uk/learning

Nursing
Times
Learning

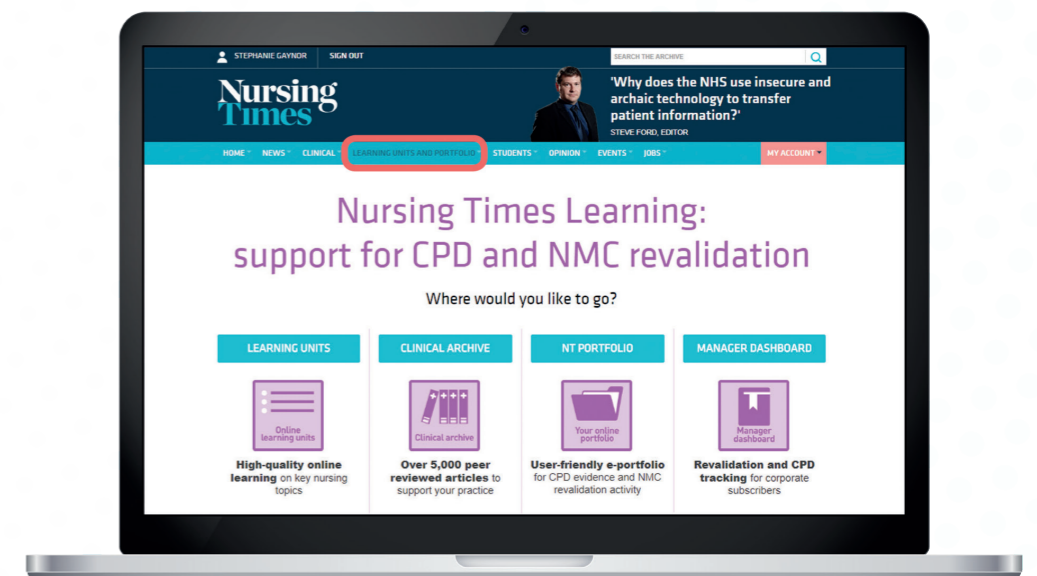

STAY CONNECTED

