

medi**pal**

soft dry wipes



Exceptionally soft & gentle
Ideal for delicate cleansing
Formulated for fragile or
sensitive skin



Made in our UK facilities
ISO 9001, 13485
& 14001 certified
Trusted quality
& compliance



Extra-large & ultra-absorbent
Tough, tear-free wipes
Fewer wipes, lower cost

Soft Dry Wipes product specification



Dimensions – 300mm x 250mm per wipe

Count – 100 wipes per pack

Primary packaging – Flow wrap 100% recyclable CPP film

Packs per case – 20

Cases per pallet – 36

Product Code – DW660110MPP

Bunzl Code – P07229

pal[®]

sales@palinternational.com | +44(0)1455 555 700 | palinternational.com



How to use

Step 1 Before you start

Pay particular attention to the elderly, children and patients with sensitive skin. Select your Medipal Patient Care Product. Use a fresh wipe for each step. Follow your standard moving and handling techniques when bed bathing patients. Do not use on wounds or broken skin. Please follow your local guidance for correct PPE usage.



Step 2 8 Step bed bathing technique



1. Head & neck

Use the wipe on face, paying particular attention to nose, ears and neck but avoid eyes.



2. Left arm & armpit

Starting from upper shoulder, clean down first arm and hand. Pay particular attention to areas between the fingers, elbow joint, and any skin folds. Clean under armpit.



3. Right arm & armpit

Starting from upper shoulder, clean down first arm and hand. Pay particular attention to areas between the fingers, elbow joint, and any skin folds. Clean under armpit.



4. Left leg & foot

Starting from hip joint, clean down first leg to the foot and sole. Pay particular attention to areas between the toes, knee joint and any skin folds.



5. Right leg & foot

Starting from hip joint, clean down first leg to the foot and sole. Pay particular attention to areas between the toes, knee joint and any skin folds.



6. Chest & abdomen

Clean the chest being careful with the sensitive areas.



7. Back & lower back

Clean the back and lower back.



8. Groin & pubic area

Wash the groin and perineal area following your standard nursing practice.

Step 3 After care

Allow the patients skin to air dry.



Pal International Limited
Unit 3 Mountpark,
Bardon II, Victoria Road,
Ellistown,
Coalville LE67 1FA, UK
E: info@palinternational.com

EU Representative:
Pal Hygiene Products Limited
Unit 5B & Unit 5H, Fingal Bay Business Park,
Balbriggan, Co. Dublin.
Republic of Ireland, K32 NY57
E: info@palinternational.com