



medipal

Detergent Wipes



3 products in this range

- Multi-surface cleaning & damp dusting
- Low-smear formulation
- Alcohol, QUAT & PHMB free



Detergent Wipes

Key Highlights

- Low-smear formula for effective cleaning and damp dusting
- Neutral pH
- Alcohol Free
- Ideal for removing soiling from surfaces prior to disinfection
- Lighter 2 litre canister with 27% plastic reduction
- Tritex material, canister, buckets, lids, and labels are fully recyclable
- Cardboard used in our outer packaging is FSC approved

How to use


STEP 1 Risk assessment

Please follow your agreed risk assessment policy guidelines regarding the use of PPE. Check medical device equipment manufacturers guidelines for DDAC and Triamine suitability.




STEP 2 Wipe selection

Check pack for use period and dispense a wipe. Close pack to protect remaining wipes. If visible soil is present, remove with a wipe. Use a fresh wipe to disinfect the surface.

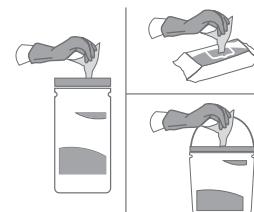


STEP 3 S-Shape technique

Wipe the surface in an S-Shape moving from clean to dirty. Use the wipe flat not scrunched, ensuring the entire surface is wetted. Do not go over the same area twice with the same wipe. Use a fresh wipe if your wipe becomes soiled or dry.

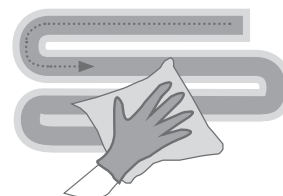


STEP 4 Discard

Do not re-use wipes. Discard used wipes in the appropriate waste bin or macerator, following your local agreed guidelines.

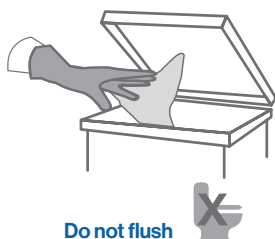

STEP 5 Let dry

Allow the surface to dry naturally before use. Leave the surface for the stated contact time.

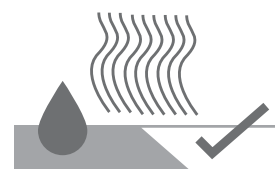

Use biocides safely. Always read the label and product information before use.

Technical details

Chemical Formulation

Formulated using environmentally friendly and VOC free ingredients

Surfactants (surface-active agents) reduce water's surface tension, allowing it to wet surfaces better, emulsify oils, and lift dirt

Dermatologically tested, pH-balanced for skin

Substrate

Pal's proprietary Tritex® material is made by bonding layers of spunbond and meltblown polypropylene fibres together in a high heat process

The addition of a hydrophilic coating means optimum and consistent liquid discharge

Tritex® is 100% synthetic

Substrate Key Features



High Strength



Excellent solution retention



Very low linting



Efficient solution release

Products within this range



Canister - 150 Wipes

Product code - W530110MP

Packs per case - 10



Bucket - 275 Wipes

Product code - W532110MP

Packs per case - 4



Bucket - 350 Wipes

Product code - W535110MP

Packs per case - 4

Manufacturer: Pal International Limited

Unit 3 Mountpark, Bardon II,
Victoria Road,
Ellistown,
Coalville,
LE67 1FA
United Kingdom

T: +44 (0) 1455 555700

E: sales@palinternational.com

W: www.palinternational.com

EU Representative:

Advena Ltd.,

Tower Business Centre
2nd Flr.
Tower Street
Swatar, BKR 4013
Malta

E: info@advena.mt

