



medipal

Spill Kit



1 product in this range

- Designed for the rapid and safe cleanup of blood and bodily fluid spills
- Contains:
 - 1 absorbent pad
 - 5 pack sporicidal disinfectant wipes



Spill Kit



Key Highlights

- Designed for rapid clean up & disinfection of body fluid spills
- Pad absorbs 1000-1200ml
- Wipes are dermatologically tested
- Wipes are PHMB & Alcohol Free
- Ready to use

How to use


STEP 1

Use appropriate PPE




STEP 2

Tear the pack open and remove the pad and the wipes

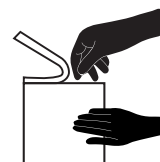


STEP 3

Unfold the pad and place on the spill to absorb the spill

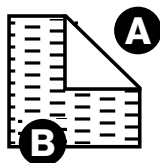


STEP 4

Push down on the pad and ensure the spill is fully absorbed

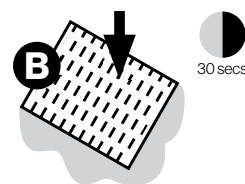


STEP 5

Safely dispose of the pad, ensuring there is no visible soiling left behind

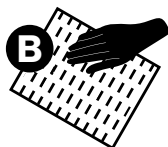


STEP 6

Remove a disinfectant wipe from the flowpack

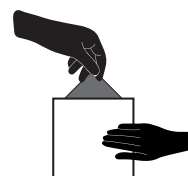


STEP 7

Disinfect the spill area using the 'S' shaped motion, wiping from clean to dirty

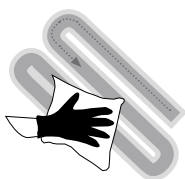


STEP 8

Safely dispose of the soiled wipe - If required, repeat step 7 and 8 as necessary

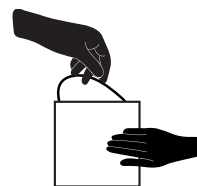

STEP 9

Reseal pack

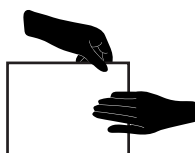

STEP 10

Safely dispose of remaining parts of the pack



Technical details

Absorbent Pad

Highly absorbent pad to contain fluid spills

Each pad absorbs up to 1200ml

Large in size for a wide range of scenarios (600 x 900mm)

Sporicidal Wipes

Medipal Chlorine Disinfectant Wipes are made with stabilised Sodium Hypochlorite (NaClO)

Thick, soft & strong spunlace wipe material

5000 PPM available Chlorine at manufacture

Pad Key Features



Hydrophilic Polypropylene Layer



High Speed Woodpulp Diversion Layer



Super Absorbent Polymer (SAP) Layer



Woodpulp Layer



PE Waterproof Layer

Medipal Chlorine Disinfectant Wipe Efficacy

Standard	Organism Group	Organism(s)	Contact Time	Conditions
EN 13727	Bactericidal	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Escherichia coli</i> K12 <i>Enterococcus hirae</i>	1 minute	Clean
EN 13727	Bactericidal	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Enterococcus hirae</i>	1 minute	Dirty
EN 16615	Bactericidal	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Enterococcus hirae</i>	1 minute	Clean
EN 16615	Bactericidal	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Enterococcus hirae</i>	1 minute	Clean
EN 14348	Mycobactericidal	<i>Mycobacterium terrae</i> <i>Mycobacterium avium</i>	1 minute	Clean
EN 14348	Mycobactericidal	<i>Mycobacterium terrae</i> <i>Mycobacterium avium</i>	1 minute	Clean
EN 13624	Fungicidal Yeast (C.a) Fungi (A.b)	<i>Candida albicans</i> <i>Aspergillus brasiliensis</i>	2 minutes	Clean
EN 13624	Fungicidal Yeast (C.a) Fungi (A.b)	<i>Candida albicans</i> <i>Aspergillus brasiliensis</i>	2 minutes	Clean
EN 17387	Fungicidal (Asp only)	<i>Aspergillus brasiliensis</i>	2 minutes	Clean
EN 13624	Yeasticidal	<i>Candida auris</i>	1 minute	Clean
EN 16615	Yeasticidal	<i>Candida auris</i>	1 minute	Clean
EN 16615	Yeasticidal	<i>Candida auris</i>	1 minute	Clean

Efficacy details

Standard	Organism Group	Organism(s)	Contact Time	Conditions
EN 14476	Virucidal (Full)	Adenovirus type 5, strain Adenoid 75 Murine norovirus, strain s99 Berlin Poliovirus type 1, strain LSc2ab	1 minute	Clean
EN 14476	Virucidal (Full)	Adenovirus type 5, strain Adenoid 75 Murine norovirus, strain s99 Berlin Poliovirus type 1, strain LSc2ab	1 minute	Clean
EN 14476	Virucidal (Human coronavirus)	Human coronavirus 229E	1 minute	Clean
EN 14476	Virucidal (Human coronavirus)	Human coronavirus 229E	15 + 30 seconds	Clean
EN 14476	Virucidal (Full)	Poliovirus type 1, strain LSc2ab	2 minutes	Dirty
		Murine norovirus, strain s99 Berlin	1 minute	Dirty
		Adenovirus type 5, strain Adenoid 75	1 minute	Dirty
EN 16777	Virucidal (Limited)	Adenovirus type 5, strain Adenoid 75 Murine norovirus, strain s99 Berlin	1 minute	Clean
EN 16777	Virucidal (Human coronavirus)	Human coronavirus 229E	30 seconds	Clean
EN 16777	Virucidal (Limited)	Adenovirus type 5, strain Adenoid 75 Murine norovirus, strain s99 Berlin	1 minute	Clean
EN 17126	Sporicidal	<i>Clostridioides difficile</i>	3 minutes	Clean
EN 17126	Sporicidal	<i>Bacillus subtilis</i> <i>Bacillus cerus</i>	5 minutes	Clean
EN 17126	Sporicidal	<i>Clostridioides difficile</i> <i>Bacillus subtilis</i>	5 minutes	Clean
EN 17126	Sporicidal	<i>Bacillus cerus</i>	8 minutes	Clean
EN 17126	Sporicidal	<i>Clostridioides difficile</i>	3 minutes	Clean
EN 17126	Sporicidal	<i>Bacillus cerus</i> <i>Bacillus subtilis</i>	5 minutes	Clean
EN 17846	Sporicidal	<i>Clostridioides difficile</i>	5 minutes	Dirty
EN 17846	Sporicidal	<i>Clostridioides difficile</i>	5 minutes	Clean

Use biocides safely. Always read the product label and information before use.

Products in this range



Spill Kit Specification:

1 x absorbent pad (600 x 900mm)

**1 x Medipal Chlorine
Disinfectant Wipes** (5 count)
(Wipe: White, 300 x 400mm)

Packs per Case - 24 Spill Kits

Product code: SP001

Manufacturer:

Pal International Limited

Unit 3 Mountpark, Bardon II,
Victoria Road,
Ellistown,
Coalville,
LE67 1FA

T: +44 (0) 1455 555700

E: sales@palinternational.com

W: www.palinternational.com

EU Authorisation Holder:

Pal Hygiene Products Limited

Unit 5B & Unit 5H,
Fingal Bay Business Park,
Balbriggan, Co. Dublin.
Republic of Ireland

T: +353 15060172

E: sales@palinternational.com

